


# Vancouver Radiologists New Web Viewer Install Instructions

Open Internet Explorer and enter <https://results.vanrad.com> into the address box.

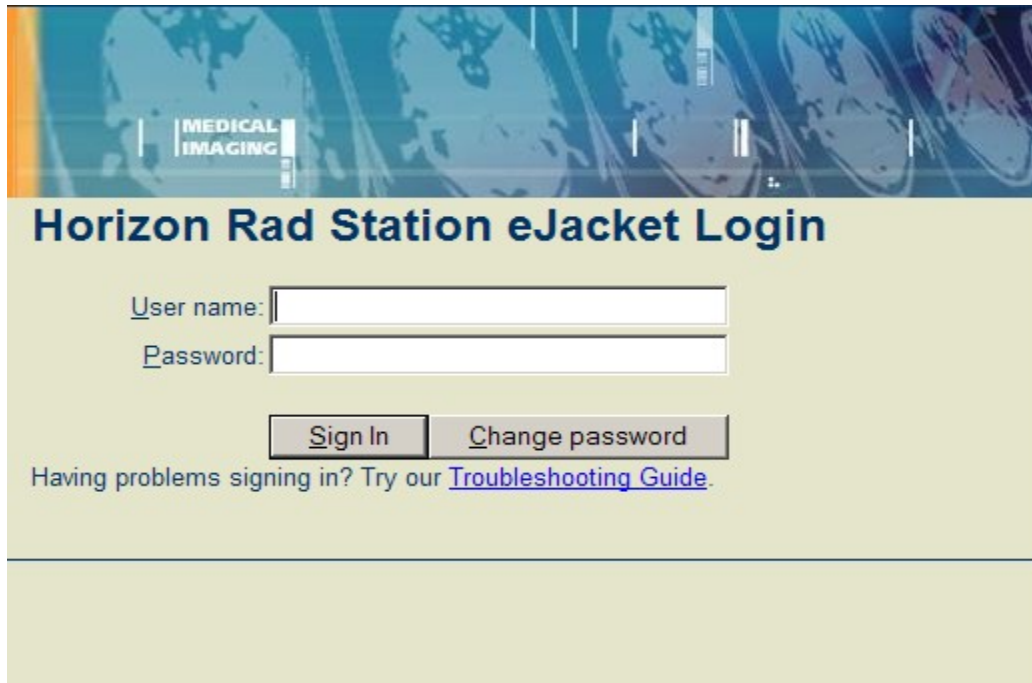

MEDICAL IMAGING


## Horizon Rad Station eJacket Login

User name:

Password:

Having problems signing in? Try our [Troubleshooting Guide](#).

This is the login screen for eJacket

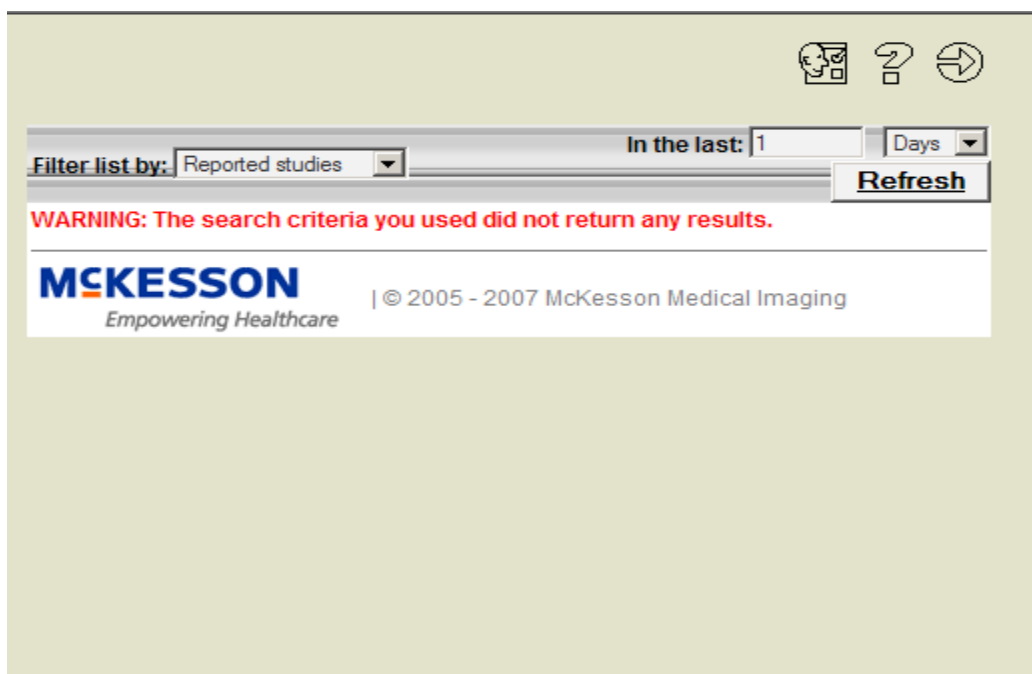

Filter list by:  In the last:  Days

**WARNING: The search criteria you used did not return any results.**


**McKESSON**  
Empowering Healthcare | © 2005 - 2007 McKesson Medical Imaging

This is the screen that you will see when you log in to eJacket. You can change the “Filter list by” drop down to “Search” or there are a few other options.


## Scenario 1:

### Installing Expanded Tools/MI View Plug-In from EJacket


1. When you are looking at an image in a study you will see the blue hyperlink that says “Launch Image Tools”

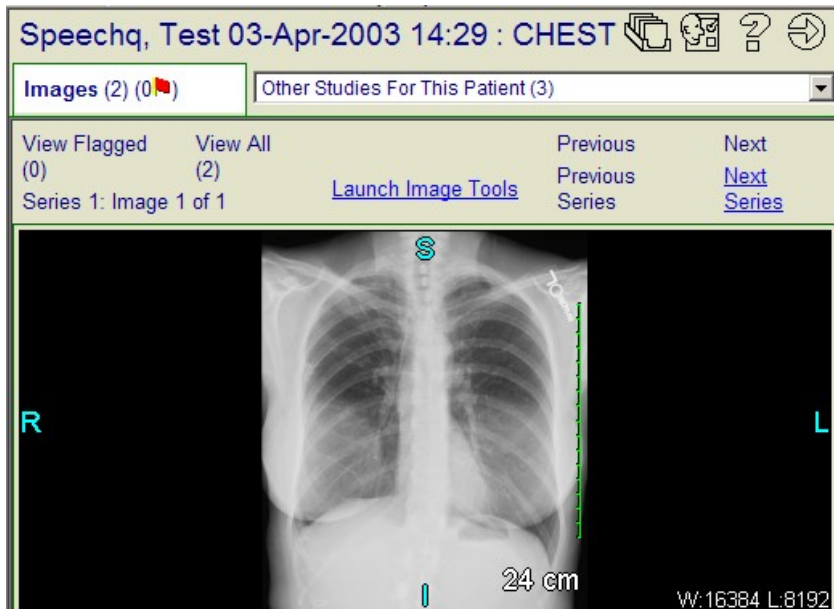


2. Go ahead and click the hyperlink to “Launch Image Tools”

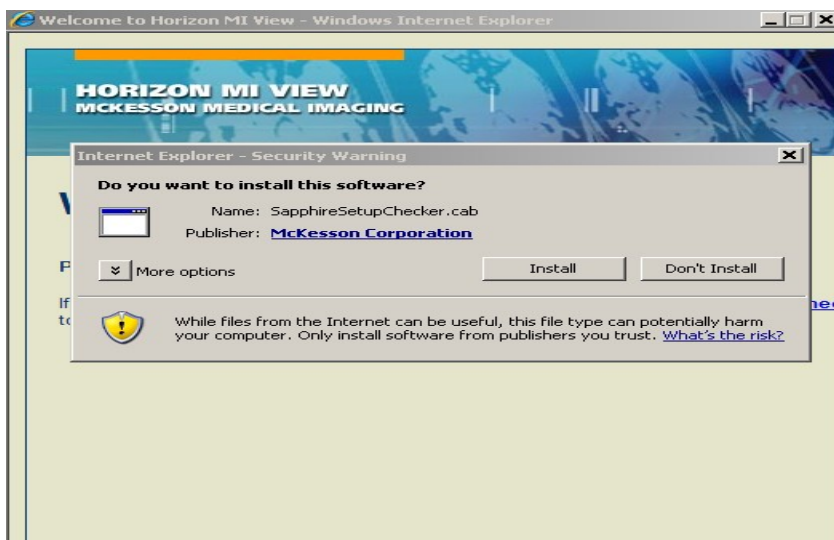


3. Click on Install

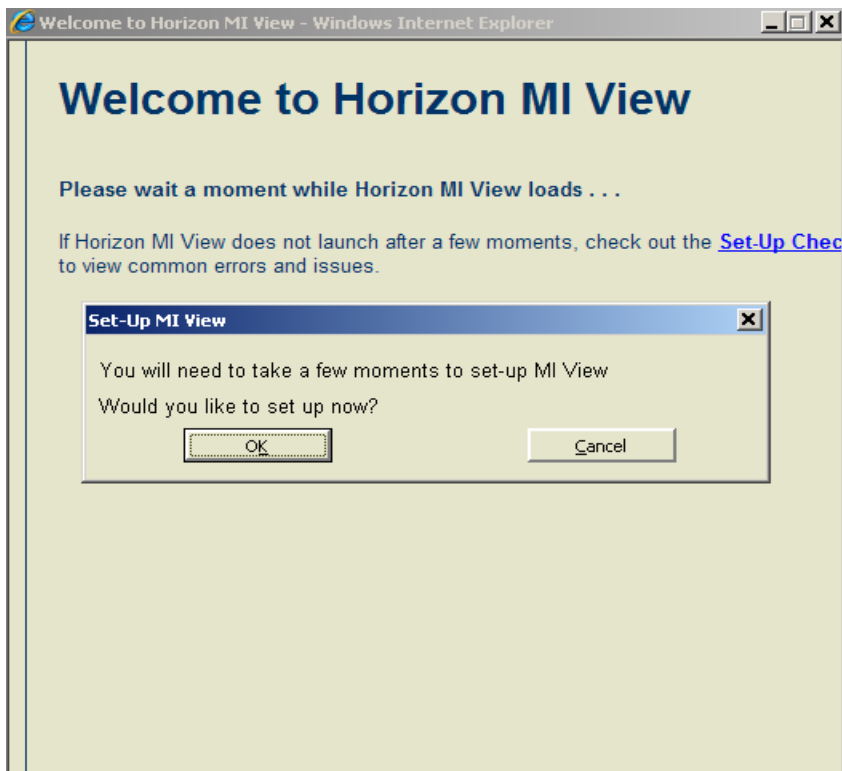


4. Click on OK

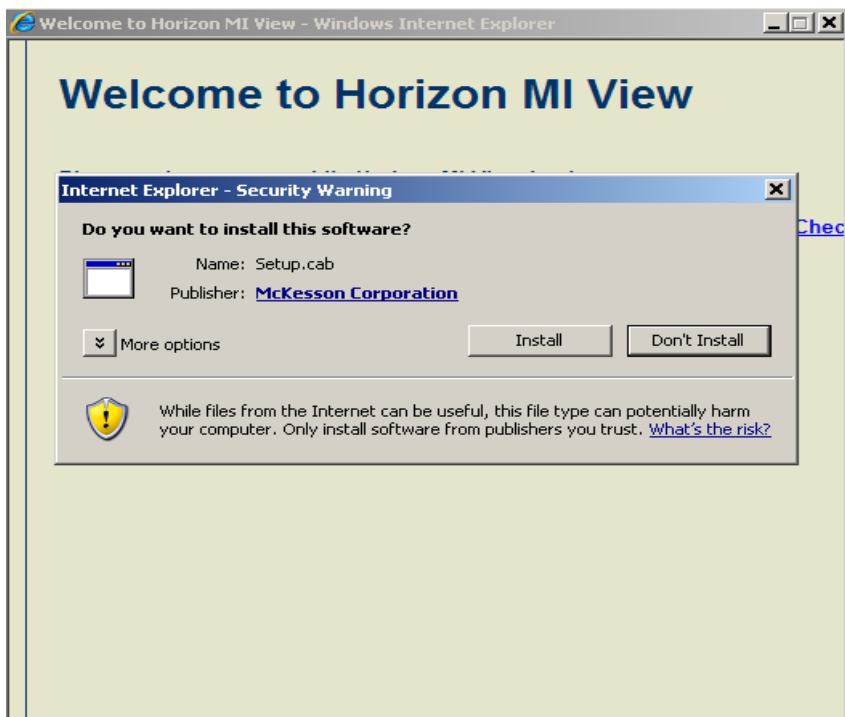


5. Click on Install

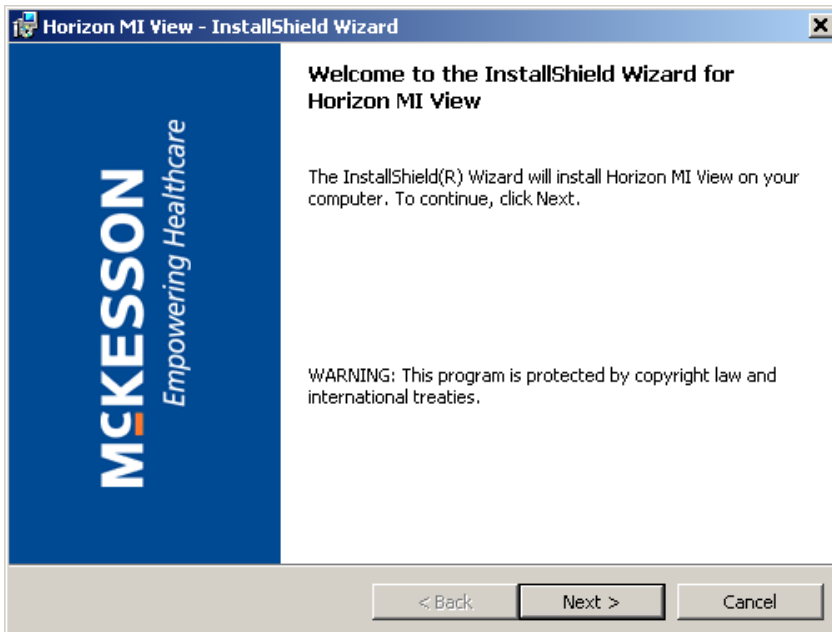

6. The install wizard now launches. Go ahead and click Next.

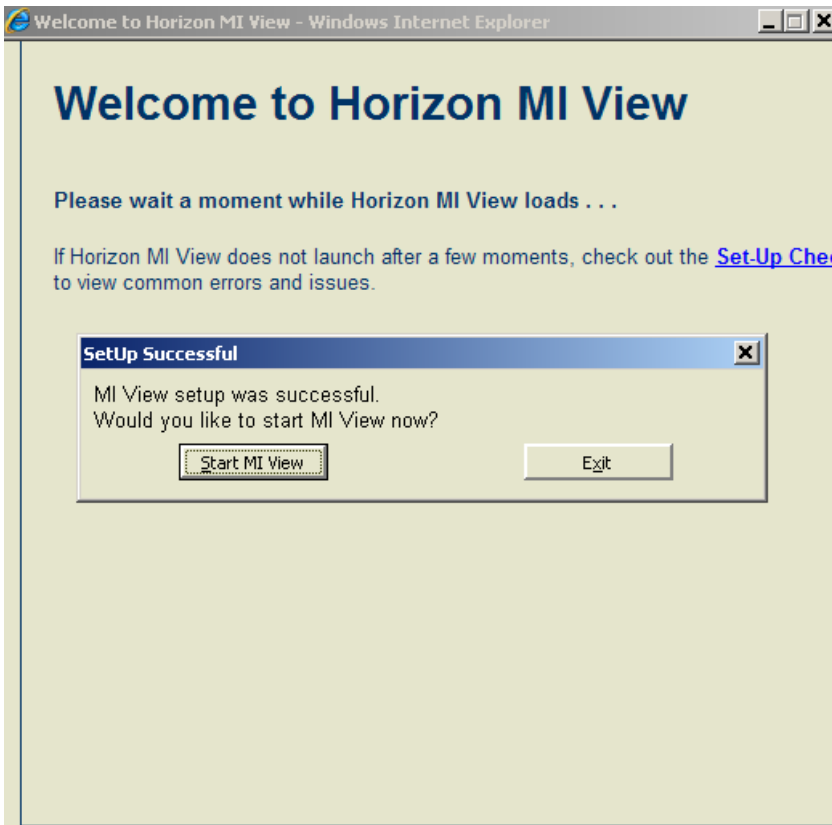

7. The install wizard has now completed. Go ahead and click on Start MI View.

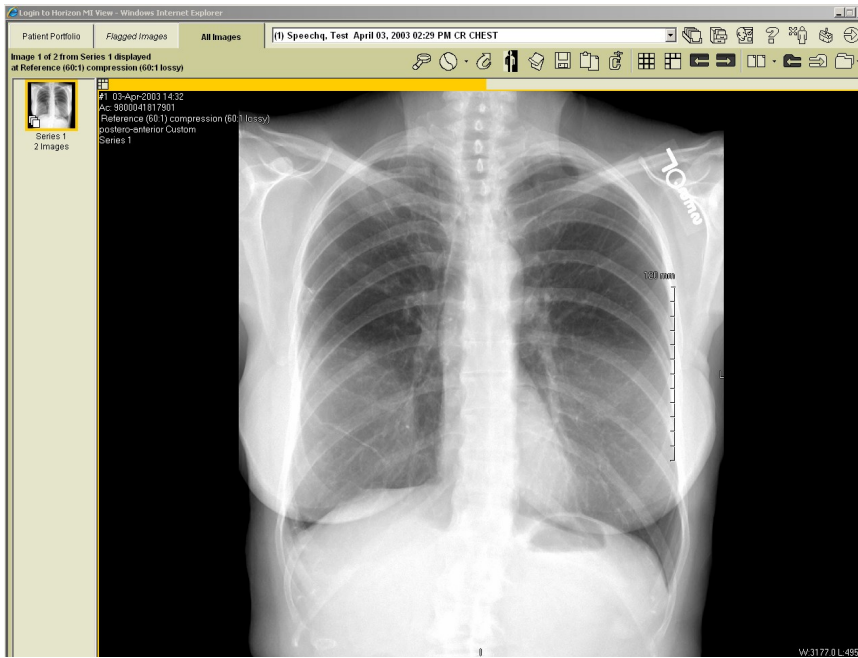



- You are now viewing the study in MI View. You can see the Image Tools at the top of the screen under the patient name and date. To view reports or flagged images, click on the tabs on the upper left hand corner of the screen.

## Scenario 2:


### Upgrading MI View when clicking on Expanded Image Tools in E Jacket

Click on Launch Image Tools

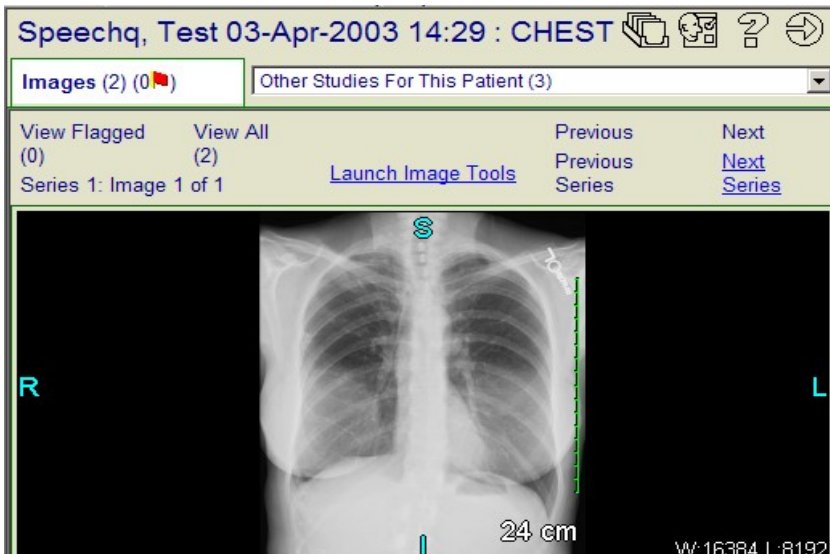


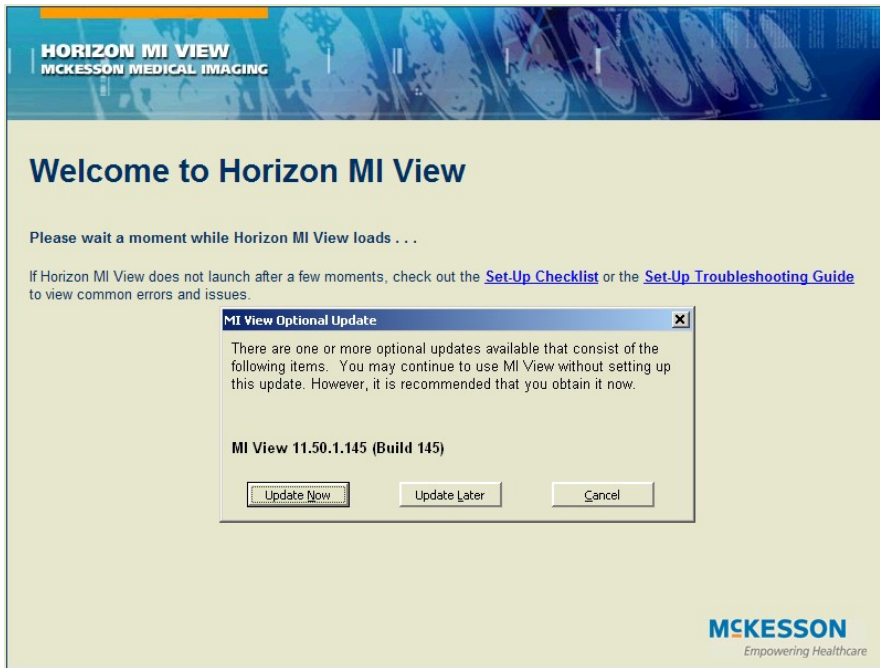

Click Update Now

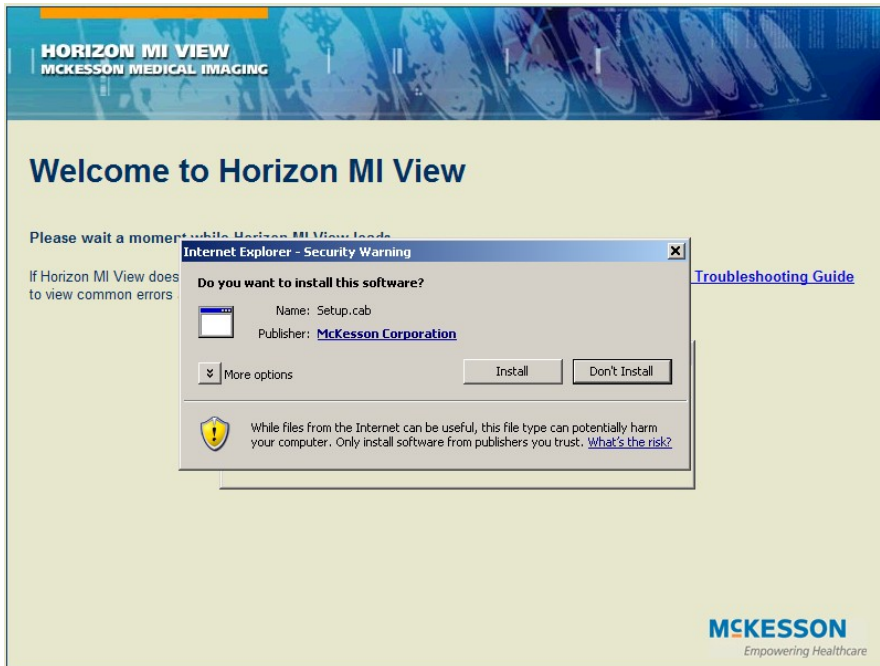


Click Install

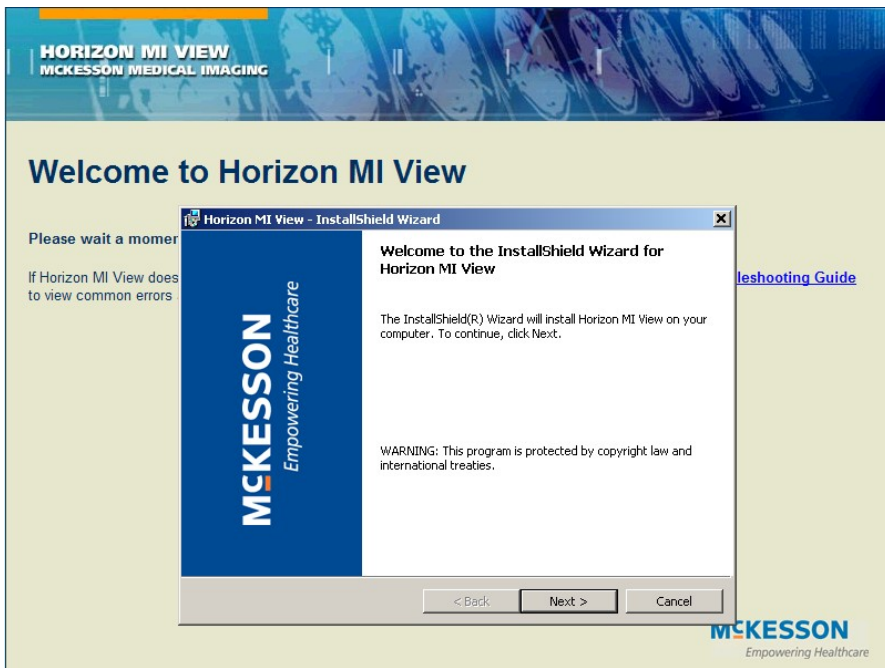


Click Next

If this window comes up for you, close your Open Internet Explorer window by clicking the X in the upper right hand corner of your screen (outlined in red below)

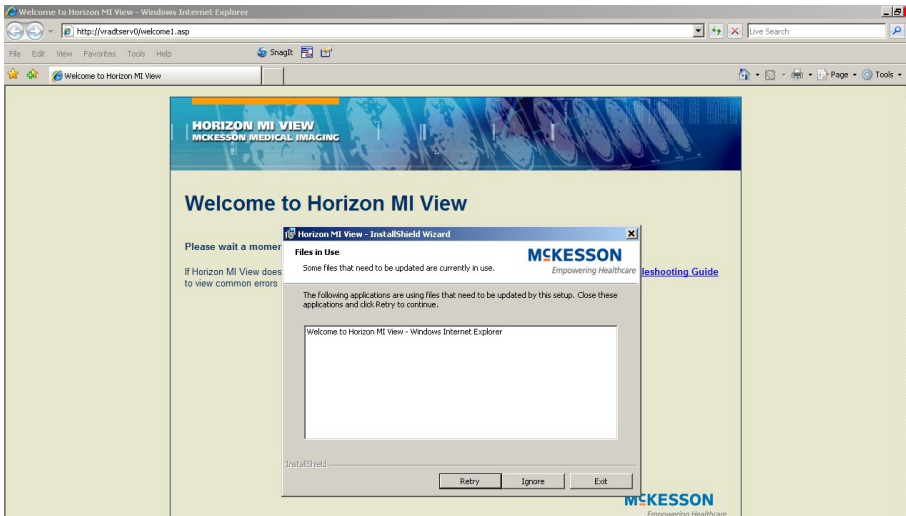


Now you should just see something like this:

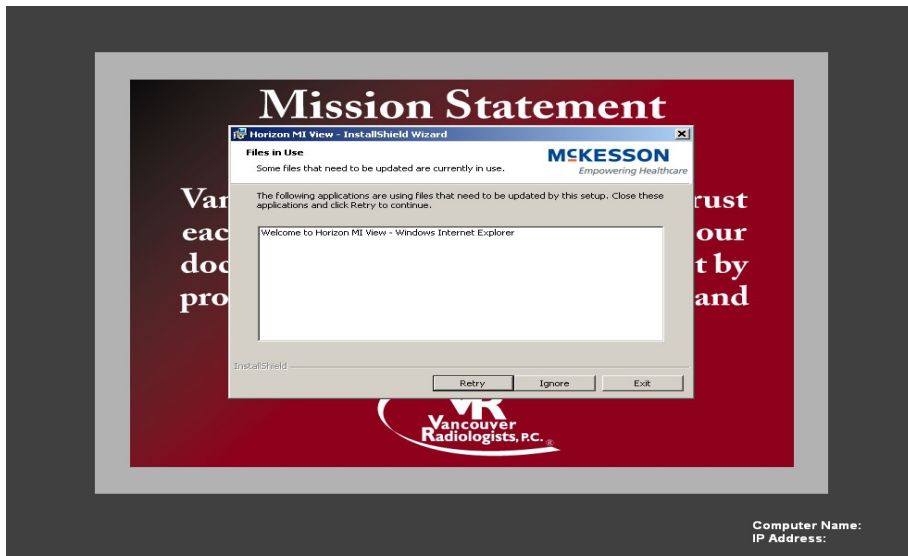

Now click on “Retry”

The install should finalize, and you will be taken back to your desktop.

Go ahead and re-open EJacket and you should be able to launch the Image Tools now.

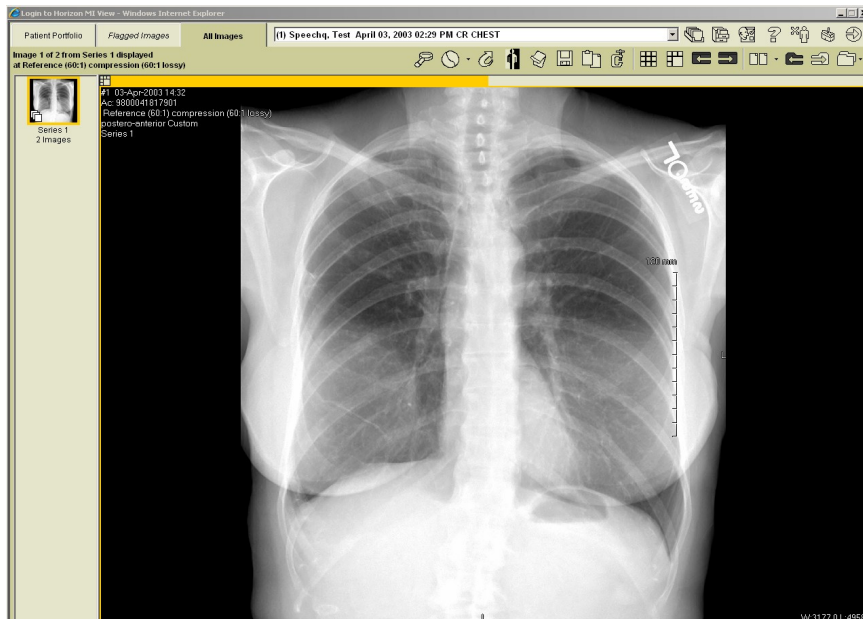

You are now viewing the study in MI View. You can see the Image Tools at the top of the screen under the patient name and date. To view reports or flagged images, click on the tabs on the upper left hand corner of the screen.

If you have any questions or concerns regarding the web viewer please utilize the following contact information:

**Vancouver Radiologists Marketing Department**

Ashley Killian, Marketing Representative: 360-449-4984

Jennifer Kammer, Director of Marketing: 360-449-4986

**Vancouver Radiologists IT department**

For limited technical support call: 360-449-4978

Our website:

<http://www.vanrad.com>

eJacket website:

<http://results.vanrad.com>